

ERGOGEL COMPLEX CUSHIONS



Description

Ergogel Complex viscoelastic and silicone gel anti-decubitus cushions.

Effects and advantages

According to the Norton scale, these cushions are recommended for patients with a high/very high risk of bedsores. Thanks to the viscoelastic and silicone material that relieves pressure on the sacrum, it distributes and adjusts the pressure in the coccyx area and the back of the buttocks. This model of cushion is anatomically shaped to facilitate the positioning of the legs.

It is essential to examine the cushion for normal wear and tear (holes, surface modifications, etc.) on a routine basis.

Suitable for use in conventional seating (chairs, armchairs, etc.), wheelchairs (with or without built-in toilet) or other means of transport for people with reduced mobility.

Silicone properties: provides greater satisfaction of freshness as well as distributing pressure evenly throughout the base of the cushion.

Maximum weight: 120kg.

Density: 90kg/m³.

Risk level: High.

Instalación

The cushion adapts to any type of seat. Use on a flat surface. Please note that the anatomical platform is for the correct positioning of the legs. For horseshoe cushions, the open part is positioned in the backrest area (see illustration). Check that the cushion is suitable for the seat where it is to be used.

Composition

- Outer cover: Two-layer fabric. 100% polyurethane lining.

- Inner part: 100% polyester fabric. Fabric thickness 230g/m².

- Core: Injected viscoelastic and silicone base. Double density of silicone in the coccyx, the area of greatest risk.

Dimensions

42x42x8cm.

Maintenance

Hand or machine washable with neutral detergent at 95°C and without spin-drying.

Do not use bleach or abrasive products.

Tumble dry at reduced temperature.

Do not iron.

IMPORTANT:

DO NOT WET THE VISCOELASTIC.




Warranty

The warranty period for all cushions except Liquid Flotation and Silicone cushions is two years from the date of purchase of the cushion. The Liquid Flotation and Silicone cushions are guaranteed for 3 months. This warranty does not cover normal wear and tear of the product and does not replace legal warranties. Please contact the company's quality department in the event of an incident in order to resolve it.

Anti-decubitus

Ergogel complex cushions

Models

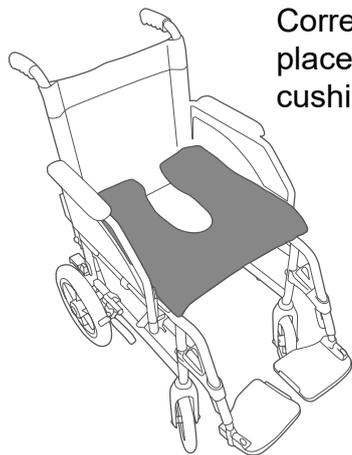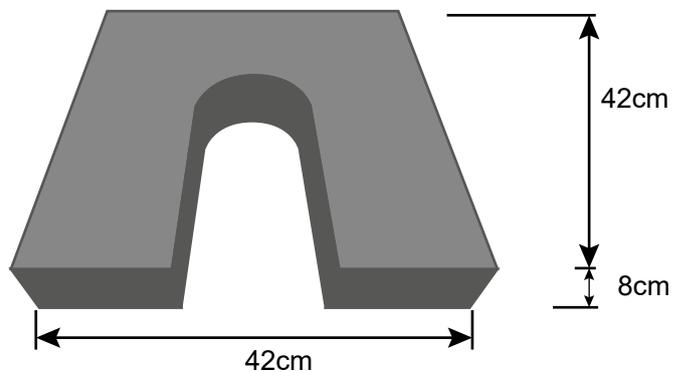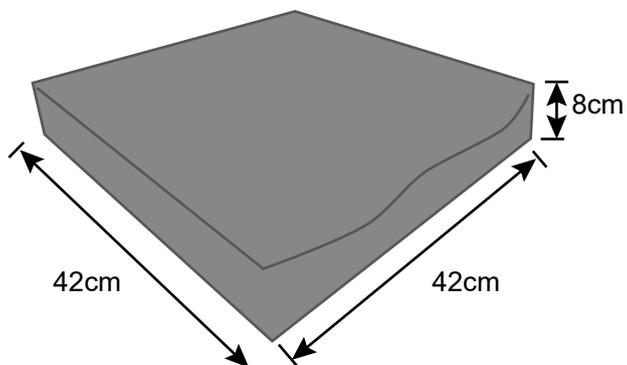

Correct way to place a horseshoe cushion.



The base of the Ergogel Complex cushion is non-slip.

Colours



Graphite

DIMENSIONS (CM)	MODEL	REF.
42x42x8	Ergogel complex cushion	102428
	Horseshoe complex ergogel cushion	102430

