

## SANILUXE CUSHIONS



### Description

Saniluxe waterproof anti-decubitus cushions, filled with conjugated silicone fibre. It has straps to fix the cushion to the wheelchair or other seats.

### Effects and advantages

Thanks to its composition and shape, the pressure area is relieved, preventing the appearance of pressure ulcers (UPP). Recommended for people who remain seated for long periods of time due to reduced mobility, sedentary life, etc. Also, depending on the pathology (postpartum, haemorrhoids, fistulas, prostate operations, etc.), one form or another is recommended to relieve pressure on the affected area.

Properties of polyurethane fabric:

Anti-mite, anti-bacterial, anti-fungal, anti-odour, anti-stain, breathable, flame retardant, waterproof, soft and washable.

Maximum support weight: 80kg.

Risk level: low-medium.

### Installation

The cushion fits any type of seat, but should be checked first. Please note that for horseshoe-shaped cushions, the open part is positioned in the backrest area (see illustration).

### Composition

- Outer cover:

Two-layer fabric:

100% polyurethane lining.

Inner part: 100% polyester fabric.

Fabric thickness: 230g/m<sup>2</sup>.

- Filling: Conjugated siliconised hollow fibre. It has 100% polyester straps to attach the cushion to the wheelchair or other seats and facilitate its position.

### Dimensions

- Round with hole: 44x11cm.

- Other cushions: 44x44x11cm.

### Maintenance

Hand wash with cold water and non-abrasive products. In case of machine washing, use a programme for delicate garments and do not spin-dry. It is recommended to use a special cover or net to avoid deformation during washing.

Dry in a horizontal position.

Do not iron.

Do not bleach.

Do not dry clean.


### Warranty

The warranty period for all cushions except Liquid Flotation and Silicone cushions is two years from the date of purchase of the cushion. The Liquid Flotation and Silicone cushions are guaranteed for 3 months. This warranty does not cover normal wear and tear of the product and does not replace legal warranties. Please contact the company's quality department in the event of an incident in order to resolve it.


# Anti-decubitus

## Saniluxe cushions


### Models

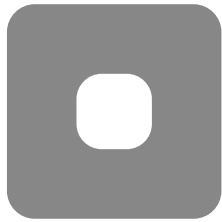

Square  
w/hole

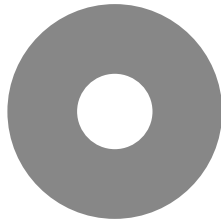


Round  
w/hole

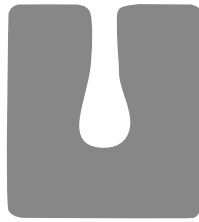

Square  
horseshoe



Round  
horseshoe

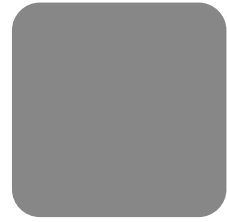

Square

### Colours



Graphite



Light  
blue



White

Correct way to  
place a horseshoe  
cushion.



MODEL	DIMENSIONS (CM)	COLOUR	VERSION	REF.
Saniluxe cushions	44x11	Graphite	Round w/hole	114100
	44x44x11	Graphite	Round horseshoe	114105
			Square horseshoe	114110
			Square w/hole	114115
			Square	114120
		White	Square	114122
		Light blue	Square	114124
		White	Square w/hole	114126
			Square w/hole	114128
		Light blue	Square horseshoe	114130
			Square horseshoe	114132
	White	Round horseshoe	114134	
		Round horseshoe	114136	
	44x11	Light blue	Round w/hole	114138
		White	Round w/hole	114140



ANTIÁCAROS



ANTI-HONGOS



ANTI-BACTERIAS



ANTI-OLOR



ANTI-MANCHAS



TRANSPIRABLE



IGNÍFUGO



IMPERMEABLE



SUAVE



LAVABLE



ANTI-DESIZANTE